

# How to use your Pulse Oximeter

Insert the supplied AAA batteries before first use of your device. After properly installing the batteries, pinch the device open.



1



Place one finger between the rubber cushions of the clip. Make sure the finger is in the correct orientation (nail facing up).

2

Press the white button to start measurement.




3



Do not remove your finger for a minimum of 30 seconds. The display will begin to display data.


4

Once you get your reading, it will display "Upload".

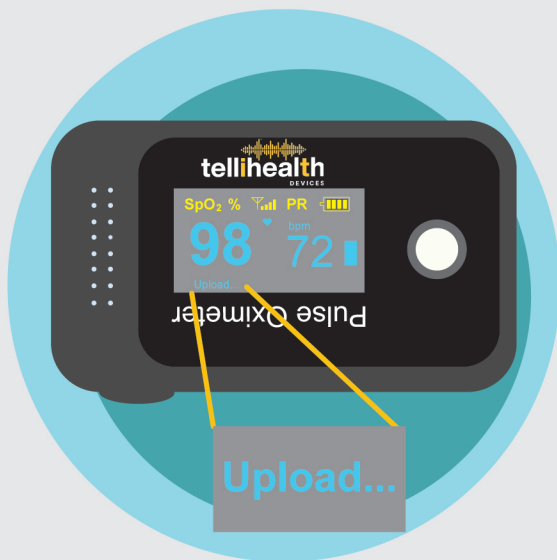

5

Remove the device from your finger and wait

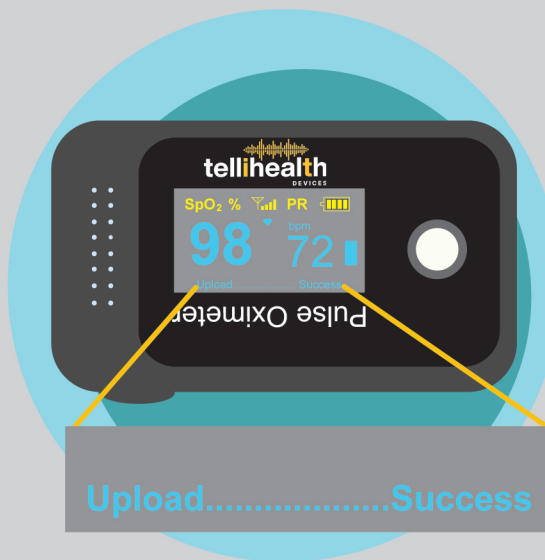

6

The device will turn off automatically after displaying "Thank you"



7